herpo therm[®] neo

Lippen-Herpes Wärmestift

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herpo therm[®] neo

Heat pen for cold sores

A medical device for symptomatic treatment of cold sores fast | simple | well-tolerated

- Overview of all benefits**
- reliable and rapid relief at the touch of a button
 - works solely with concentrated heat (Thermo Sense Technology)
 - if applied early enough, the formation of cold sore blisters can be prevented
 - very well tolerated – herpo therm[®] neo works without chemicals and is suitable for allergy sufferers, pregnant women and children from 3 years of age
 - can be powered with either non-rechargeable or rechargeable batteries
 - patented multi-level safety features:
 1. intelligent temperature control
 2. independent temperature guard
 3. fail-safe overheating protection


Please read the entire instruction for use carefully before the first application and follow the instructions and warnings.

Intended use
herpo therm[®] neo is a patented medical device for local treatment of the accompanying symptoms of herpes (herpes simplex labialis), such as pain, burning, itching, tingling or a feeling of tightness. These so-called prodromal symptoms can already indicate the start of a herpes outbreak before external signs appear. If herpo therm[®] neo is applied early enough, the formation of cold sore blisters can be prevented. If applied later, the severity of symptoms may be lessened, or they may subside sooner.

How it works
The application of concentrated heat (local thermotherapy) is a physical mode of action based on the application of a brief, concentrated thermal stimulus to a small, limited area of skin. Even a local application of heat may be sufficient to relieve discomfort such as itching, burning, tingling or a feeling of tightness and, if applied early enough, to prevent the formation of cold sore blisters.

In a clinical study with herpo therm[®], it was shown that for those affected the symptoms accompanying herpes were reduced significantly and clinically relevant within 24 to 48 hours by treatment with a heat pulse.

Application
Design of herpo therm[®] neo:

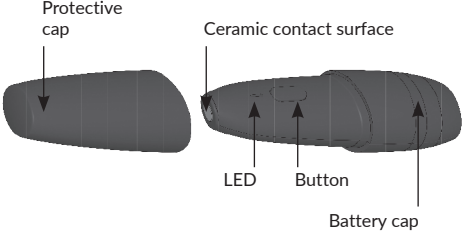Initial commissioning:
The packaging of herpo therm[®] neo includes two AAA LR03 1.5 V batteries. The device is delivered with the batteries inserted. You can start to use it straight away.

Treatment:

herpo therm[®] neo is very simple to use:

1. Remove the protective cap from the device.
2. Place the ceramic contact surface of the device onto the affected skin area and gently press it against the skin.
3. The heating phase starts after the button is pressed. During the heating phase, the ceramic contact surface heats up to a temperature of approximately 51 °C. Once this temperature is reached, the treatment of 3 seconds starts. The LED lights up during treatment. The device switches off automatically after treatment. An additional acoustic signal sounds at the start of the heating phase and at the end of treatment.
4. Remove the device from the area of skin treated and place the protective cap back on the device.

If symptoms persist or recur, the same skin site can be treated again. Please leave a treatment interval of at least 2 minutes between each application. Subsequent treatment of another affected area of skin on the lip immediately after the first treatment with herpo therm[®] neo is possible without any problems.

Directions for use

herpo therm[®] neo is indicated for self-administration by adults and children from 12 years of age. For children under 12 years of age, the device should only be used with adult supervision. Person with a limited ability to perceive the visual or acoustic signals of the device should be assisted by an adult when using the device. Children under 7 years of age are often not yet able to understand the connection between a short heat treatment and its positive benefits. Please therefore check carefully whether symptomatic treatment with herpo therm[®] neo is suitable for your child. Please also bear in mind that children's skin is still developing and, thus, can react more sensitively to heat stimuli. Do not use the device in children under 3 years of age.

Cleaning and disinfection:
Herpes is a viral disease that is caused by the herpes simplex virus. Transmission is via droplet infection, through direct contact or as a smear infection. An infection with the herpes simplex virus cannot be reversed. To prevent virus transmission, herpo therm[®] neo shall only ever be used by one person.

Clean herpo therm[®] neo by wiping all external surfaces with a soft cloth slightly moistened with water. If the device is obviously contaminated, it must be cleaned. To do this, please use a soft cloth moistened with 70% isopropanol. Wipe all external surfaces thoroughly until no more contamination is visible. Then dry these surfaces with a cloth.

You must disinfect herpo therm[®] neo after each herpes episode. In addition, it is recommended to disinfect the device before using it again after a prolonged period of non-use. To do this, moisten a soft cloth well with 70% isopropanol or use a conventional disinfecting cloth based on 70% isopropanol. Wipe the ceramic contact surface thoroughly with this cloth for 30 seconds. Allow the surface to air dry.

Changing the batteries:

Figure 1

Figure 2



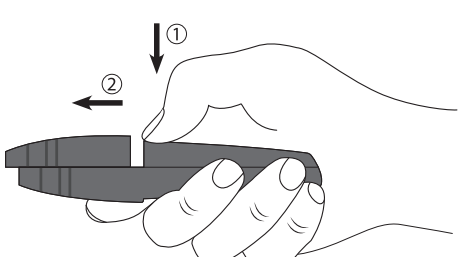
Figure 3

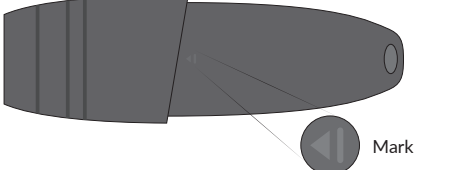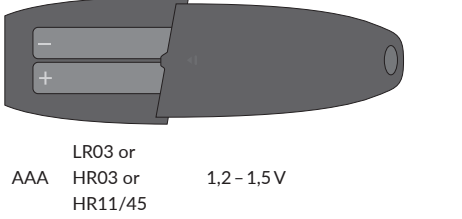herpo therm[®] neo can be used with either non-rechargeable or rechargeable batteries. To change the batteries, first remove the protective cap (Figure 1). Then open the battery cap (Figure 2) by pressing lightly on the mark with your thumb ① while sliding the battery cap in the direction of the arrow ②.

Remove both discharged batteries. Insert the new batteries into the device observing the indicated polarity (Figure 3). After inserting the batteries, the battery cap must be pushed back on completely along the guides. When the batteries need to be replaced, the LED will flash 3 times and the acoustic signal will sound 3 times. Please use batteries of type AAA LR03, HR03 or HR11/45 with a voltage of 1.2 V to 1.5 V. Please note that using rechargeable batteries with a voltage of 1.2 V may result in a lower number of applications per battery charge.

Signals and troubleshooting:

LED flashes 3 x acoustic signal 3 x	– Treatment temperature was not reached. – Batteries too low: Change the batteries (see section "Changing the batteries").
LED flashes 5 x acoustic signal 5 x	– Malfunction: The cause can be incorrect use, e.g. triggering and operating the device against air. – If this error message occurs repeatedly despite proper use, please contact the manufacturer or distributor of herpo therm [®] neo.
herpo therm [®] neo does not heat up/ does not work	– Change the batteries (see section "Changing the batteries"). – If this error message occurs repeatedly despite fully charged batteries, please contact the manufacturer or distributor of herpo therm [®] neo.

Warnings

herpo therm[®] neo must not be used on or in cases of

- eyes or eyelids
- mucous membranes
- open wounds or damaged skin (e.g. open blisters)
- scalded or burnt skin
- frostbite
- tumours and malignant changes of the skin
- known increased algesia (increased pain sensation) in small fibre neuropathy, allodynia, hyperalgesia
- people with known limited sensitivity to pain and temperature.

Medical advice should be sought in the following cases:

- if symptoms persist when using herpo therm[®] neo.
- if symptoms worsen when using herpo therm[®] neo. Until these have been clarified, you should stop using the device.

Furthermore, the device must not be used immediately after the application of ointments, creams, lotions and other preparations on the skin area to be treated as possible interactions with ingredients of these products cannot be ruled out.

In individual cases, the treatment temperature may cause some discomfort. On sensitive skin, a temporary reddening of the skin may occur after use. In very rare cases, skin irritation may occur in users with a sensitive skin type.

Do not use the device in case of damage, obvious malfunction or with open housing. Do not use damp or damaged batteries. Protect the device from moisture. Any changes to herpo therm[®] neo are not permitted. Store the device out of reach of children. Please note that herpo therm[®] neo contains small parts that could be swallowed by children under 3 years of age and, thus, poses a choking hazard.

Permissible environmental conditions

Store the device protected from direct sunlight, in a dry and clean place.

Please observe the following permissible environmental conditions:

	Temperature [°C]	Relative humidity [%]	Air pressure [hPa]
Operation			
Transport*	5 to 40	15 to 85	700 to 1060
Storage*			

* Please note that the manufacturer of the batteries may have stated different storage conditions.

Please observe the handling instructions on the batteries you use. Only combine batteries of the same type and state of charge.

Do not use the batteries beyond the expiry date stated on them. To prevent the risk of damage to herpo therm[®] neo due to leaking battery acid, remove the batteries from the unit if it is not going to be used for a prolonged period of time.

herpo therm[®] neo has an expected service life of at least 5 years.

Remove the used batteries and dispose of them according to the local regulations for the disposal of batteries.

The device must not be disposed of in household waste. Dispose of the device according to local regulations for disposal of waste electrical equipment.

Manufacturer:
mibetec GmbH
Münchener Straße 15 | 06796 Brehna | Germany
e-mail: contact.mibetec@dermapharm.com

www.herpotharm.com

Please contact the manufacturer or distributor if you have any questions about the product or how it works. Please report any serious incident that has occurred in relation to herpo therm[®] neo to the manufacturer and the competent authority in your country.

Date of revision of the text: 2025-10

This medical device complies with the EMC standard IEC 60601-1-2:2014+A1:2020.

Nevertheless, special precautions should be taken: herpo therm[®] neo is intended for use in home healthcare environment.

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation.

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm to any part of herpo therm[®] neo, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

According to the specifications of IEC 60601-1-2:2014+A1:2020 the technical description can be requested in digital form in English or German from mibetec GmbH (address can be found in the section "Manufacturer").

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**mibetec** GmbH